



PLEASE READ THIS DOCUMENT BEFORE INSTALLING AND USING THE PRODUCT!



HRV LIVE

Quick Setup Guide

Note: All instructions in this section imply you are using the Windows 11 operating system. Please look for equivalent options if you have a different Windows version.

Non-medical Disclaimer. This instrument is not designed or intended for rendering medical advice or being used for providing professional medical services. The information it provides should not be used to diagnose or treat any health problem or disease, nor is it a substitute for professional care.

1. Check HRV Live Box Content

When you open your **HRV Live** package, it must contain the following:

1. Selected ECG recorder with relevant accessories
2. Software installation on a USB flash drive
3. Printed insert card providing information on how to access this setup guide and other instructions for this product.

2. Minimum Computer Requirements

The following minimum requirements are applied to run **HRV Live** software:


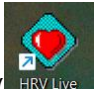
CPU:	Intel i5 (or equivalent)
RAM:	8 Gb
Video card:	Capable of at least 1024 x 768 and 32-bit color resolution
Free disk space:	2 Gb
USB port:	1 port, version 2.0
OS:	Windows 10 / 11 (<i>older versions are not supported</i>)



PLEASE READ THIS DOCUMENT BEFORE INSTALLING AND USING THE PRODUCT!

3. Install the HRV Live Software

Follow these steps to install the **HRV Live** software:

- STEP 1:** Insert the USB flash drive into a USB port.
- STEP 2:** Click  (File Explorer icon) on the Windows taskbar.
- STEP 3:** Select **This PC** on the left panel.
- STEP 4:** Select the **USB Drive**.
- STEP 5:** Find the **HRVL_Install.exe** file, right-click on it, and select **Run as administrator**.
- STEP 6:** When the User Access Control (UAC) prompts to confirm software installation, click **Yes** to continue.
- STEP 7:** Follow the on-screen instructions to complete the software installation.
- STEP 8:** Find the **HRV Live** () shortcut icon on the desktop screen.
- STEP 9:** Right-click on it and select **Properties**.
- STEP 10:** Select the **Compatibility** tab.
- STEP 11:** Set the checkmark to **Run this program as an administrator**.
- STEP 12:** Click **OK**.